




# UNILINE® Remote control unit cover

**STERILE R**


**Intended purpose:** Sterile remote control cover for the duration of one application

1

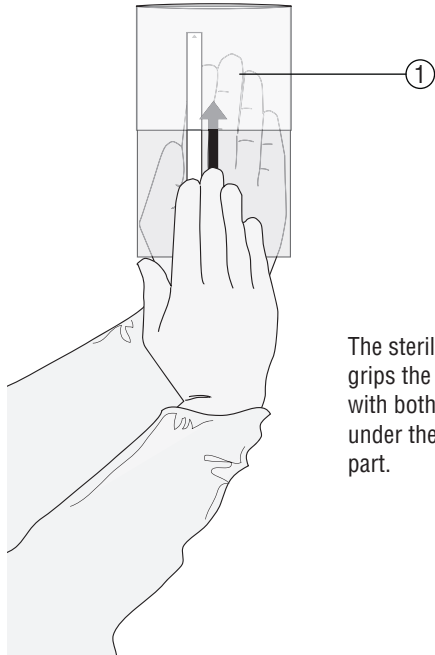


The sterile person grips the cover (1) with both hands under the folded part.

2

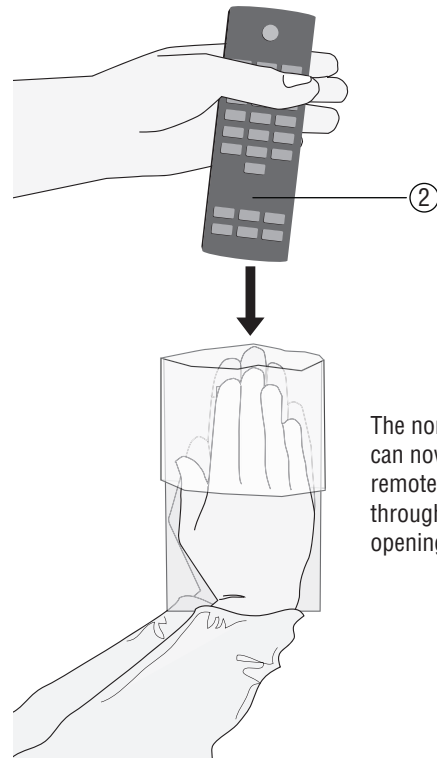


The non-sterile person can now insert the remote control (2) through the widened opening.

3

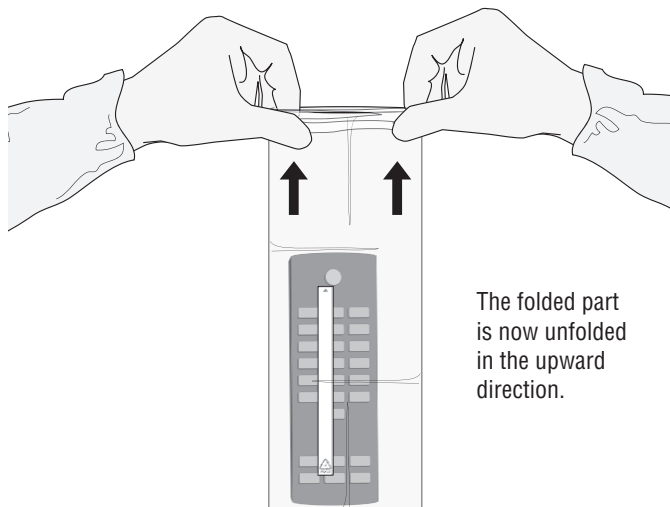

The folded part is now unfolded in the upward direction.

4

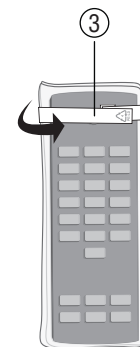









Any protruding parts of the cover are folded back and down and fixed with the adhesive tape (3).

















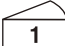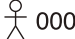


PACKAGE CONTENT		DETAILS
<b>REF 30002</b>	UNILINE® Remote control unit cover	Size: 160 x 350 mm
A detailed product specification is available on request.		

## WARNINGS and IMPORTANT INFORMATION

-  Do not use if sterility is compromised or if the packaging is damaged
-  **CAUTION: Do not use damaged products!**
-  Do not reuse
-  Do not resterilize
-  **CAUTION: Not suitable for reuse**, as the required features may fail and cause an infection of the patient.
-  **CAUTION: The instructions for use apply only to the enclosed products.**
-  **CAUTION: Open peel pouch at top seam!**
-  Store in a dry place
-  Keep away from direct sunlight
-  **CAUTION: Using a device cover may impair the quality of the image.**
-  **ATTENTION: The use of a device cover can make visual analysis more difficult!**
-  **CAUTION: To be used only by healthcare professionals!**
-  **CAUTION: Any serious incident that has occurred in relation to the device has to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.**

## OTHER SYMBOLS USED

-  Sterilized using irradiation
-  Use-by date
-  Latex-free
-  Consult instructions for use. The latest instructions for use can be downloaded at [www.heisig.de](http://www.heisig.de)
-  Number of consumption units per peel pouches
-  Personnel number of the manufacturing employee
-  Batch code  
JJMM0123
-  Manufacturer
-  Number of peel pouches in a carton
-  Date of manufacture
-  Article number  
REF 00000
-  Pharmacy central number  
PZN: 0000000
-  The product is compliant with valid legal requirements for medical devices. Manufacturer is monitored by the notified body with number 0197.
-  Medical device
-  Single *sterile* barrier system
-  Single *sterile* barrier system with protective packaging outside
-  Environmentally compatible if properly disposed of. No toxic gases arise from burning. Dump without groundwater pollution.

