




# STERINORM® Bottom bag

STERILE	R
---------	---


**Intended purpose:** Sterile cover for covering equipment for the duration of one application

1

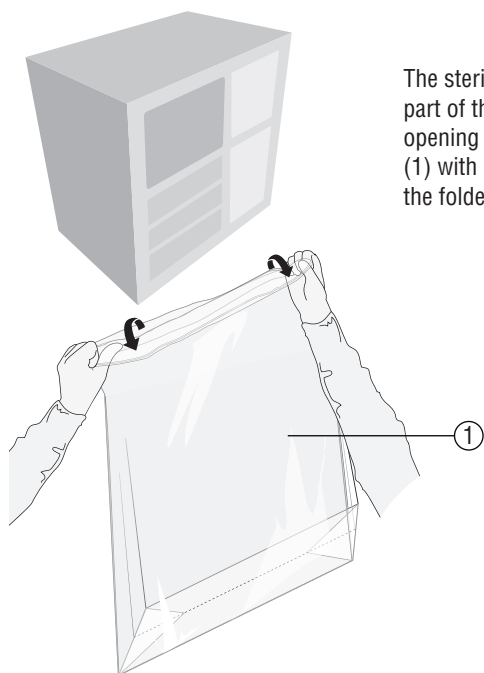

The sterile person folds a part of the bag at the bag opening and grips the cover (1) with both hands inside the folded part.

2

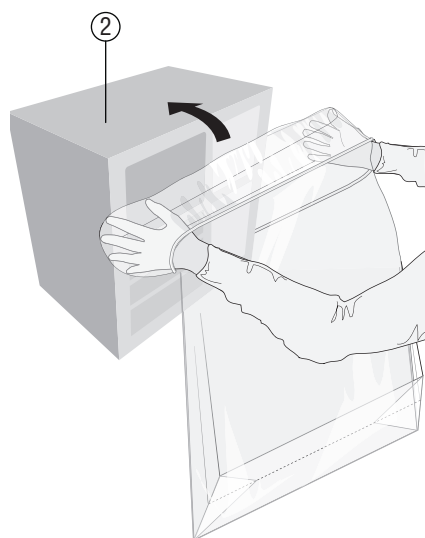


The bag is now pulled over the device (2) through the widened opening.

3



The device is completely covered with the cover.

4

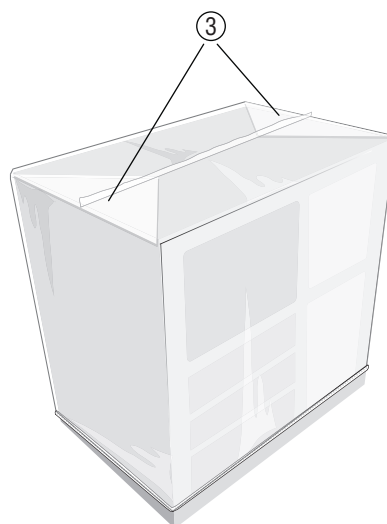

The flat-bottom folding (3) prevents protruding film areas.



VARIANTS						
STERINORM® Bottom bag	REF 30061	REF 30077	REF 30079	REF 30086	REF 30094	REF 30096
Size	Length: 560 mm Bottom: 450 x 170 mm	Length: 570 mm Bottom: 460 x 240 mm	Length: 600 mm Bottom: 490 x 300 mm	Length: 850 mm Bottom: 490 x 300 mm	Length: 550 mm Bottom: 600 x 400 mm	Length: 830 mm Bottom: 600 x 400 mm
A detailed product specification is available on request.						

## WARNINGS and IMPORTANT INFORMATION



Do not use if sterility is compromised or if the packaging is damaged



**CAUTION: Do not use damaged products!**



Do not reuse



Do not resterilize



**CAUTION: Not suitable for reuse**, as the required features may fail and cause an infection of the patient.



**CAUTION: The instructions for use apply only to the enclosed products.**



**CAUTION: Open peel pouch at top seam!**



Store in a dry place



Keep away from direct sunlight



**CAUTION: Using a device cover may impair the quality of the image.**



**CAUTION: Ventilation openings should not be covered under any circumstances!** Covered devices may overheat.



**CAUTION: To be used only by healthcare professionals!**



**CAUTION: Any serious incident** that has occurred in relation to the device has to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## OTHER SYMBOLS USED



Sterilized using irradiation



Use-by date



Latex-free



Consult instructions for use. The latest instructions for use can be downloaded at [www.heisig.de](http://www.heisig.de)



Number of consumption units per peel pouches



Personnel number of the manufacturing employee



Batch code  
JJMM0123



Manufacturer



Number of peel pouches in a carton



Date of manufacture



Article number



Pharmacy central number



The product is compliant with valid legal requirements for medical devices. Manufacturer is monitored by the notified body with number 0197.



Medical device



Single *sterile* barrier system



Single *sterile* barrier system with protective packaging outside



Environmentally compatible if properly disposed of.  
No toxic gases arise from burning.  
Dump without groundwater pollution.

