eIFU

**UDO HEISIG GMBH**  
 THE DISPOSABLES COMPANY

# FLEXANORM® Formbezug C-Set **STERILE R**

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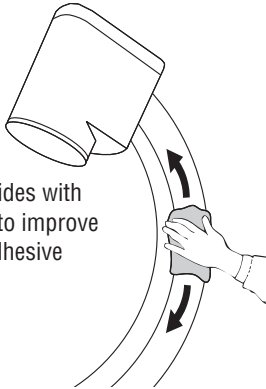
**Intended purpose:** Sterile C-arm cover for the duration of one application



1

### Before use:


Clean the C-arm sides with isopropyl alcohol to improve adhesion of the adhesive surfaces.

2

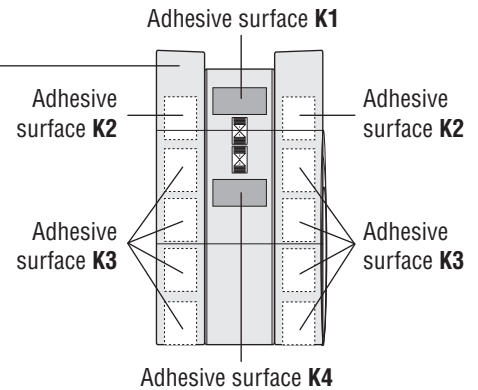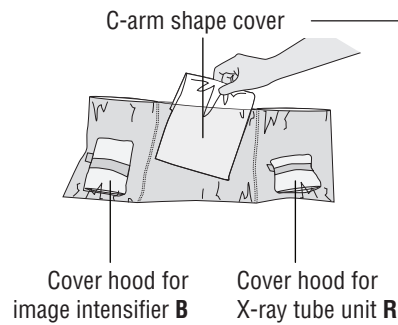
3

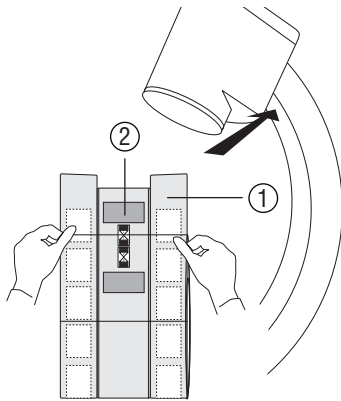
Take the C-arm package (1) with both hands in the middle and remove the blue protective paper from the adhesive surface **K1** (2).

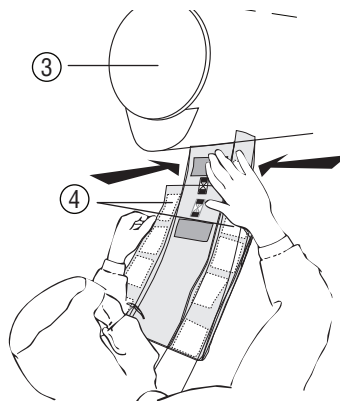4

Attach the C-arm package with adhesive surface **K1** underneath the image intensifier (3). Remove the protective paper of the adhesive surfaces **K2** (4) and attach the cover to the sides of the C-arm.

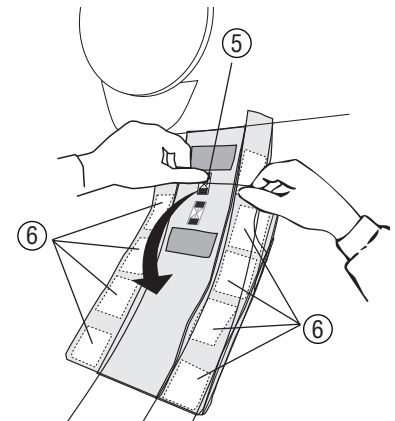
5

Open the arrow label **1** (5) and fold down the foil part. Remove the protective paper from the adhesive surfaces **K3** (6) and attach the cover to the sides of the C-arm.


6

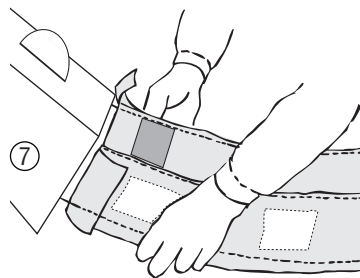Proceed in the same way with arrow label **2**, unfold the shaped part completely and attach it in front of the X-ray tube (7) and to the sides of the C-arm using the adhesive surface **K4**.

7

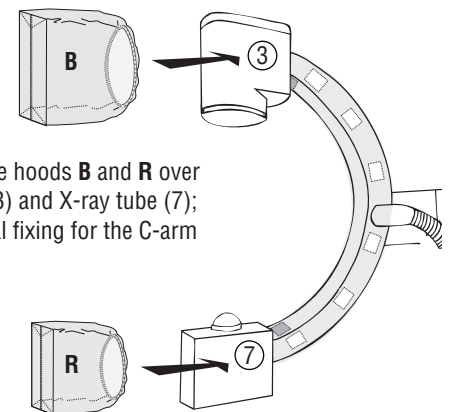Pull the attached sterile hoods **B** and **R** over the image intensifier (3) and X-ray tube (7); this provides additional fixing for the C-arm cover.


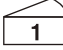
		COMPONENTS							
		SHAPE COVER for C-arm	COVER HOOD (Bottom Bag) for image intensifier (B)					COVER HOOD (Bottom Bag) for X-ray tube unit (R)	
		REF 34500	REF 34700	REF 30054	REF 30077	REF 30079	REF 30096	REF 30086	REF 30061
		Length: 165 cm	Length: 181 cm	Size: 40(+17) x 60 cm	Size: 46(+24) x 69 cm	Size: 49(+30) x 75 cm	Size: 60(+40) x 103 cm	Size: 49(+30) x 100 cm	Size: 45(+17) x 65 cm
SETS 3 parts	REF 34521	●		●					●
	REF 34531	●			●				●
	REF 34541	●				●			●
	REF 34551	●					●		●
	REF 34561	●						●	●
	REF 34731		●		●				●
	REF 34741		●			●			●
	REF 34751		●				●		●

A detailed product specification is available on request.

### WARNINGS and IMPORTANT INFORMATION

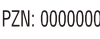-  Do not use if sterility is compromised or if the packaging is damaged
-  **CAUTION: Do not use damaged products!**
-  Do not reuse
-  Do not resterilize
-  **CAUTION: Not suitable for reuse**, as the required features may fail and cause an infection of the patient.
-  **CAUTION: The instructions for use apply only to the enclosed products.**
-  **CAUTION: Open peel pouch at top seam!**
-  Store in a dry place
-  Keep away from direct sunlight
-  **CAUTION: To be used only by healthcare professionals!**
-  **CAUTION: Any serious incident** that has occurred in relation to the device has to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### OTHER SYMBOLS USED

-  Sterilized using irradiation
-  Use-by date
-  Latex-free
-  Consult instructions for use. The latest instructions for use can be downloaded at [www.heisig.de](http://www.heisig.de)
-  Number of consumption units per peel pouches
-  Personnel number of the manufacturing employee
-  Batch code  
JMMM0123
-  Manufacturer
-  Number of peel pouches in a carton
-  Date of manufacture
-  Article number
-  Pharmacy central number
-  The product is compliant with valid legal requirements for medical devices. Manufacturer is monitored by the notified body with number 0197.
-  Medical device
-  Single *sterile* barrier system
-  Single *sterile* barrier system with protective packaging outside
-  Environmentally compatible if properly disposed of. No toxic gases arise from burning. Dump without groundwater pollution.

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