STERINORM® Bodennaht-Beutel

STERILE R

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
STERINORM® Bag with bottom seam

STERILE R


Intended purpose: Sterile cover for covering equipment for the duration of one application

1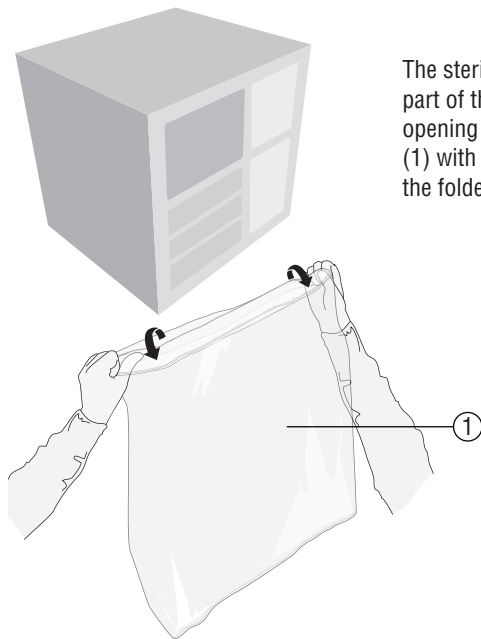

The sterile person folds a part of the bag at the bag opening and grips the cover (1) with both hands inside the folded part.

2

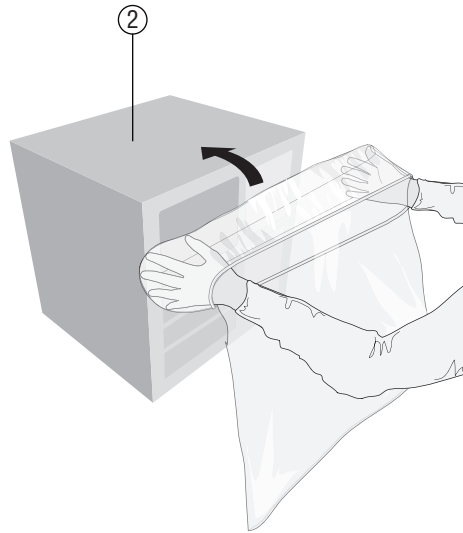
The bag is now pulled over the device (2) through the widened opening.

3

The device is completely covered with the cover.

4

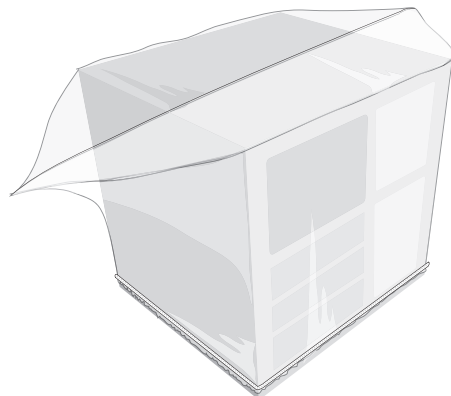VARIANTS							
STERINORM® Bag with bottom seam	REF 30045	REF 30065	REF 30085	REF 30160	REF 30180	REF 30200	REF 30250
Size	800 x 450 mm	800 x 650 mm	800 x 850 mm	1000 x 600 mm	1000 x 800 mm	1000 x 1000 mm	1000 x 1500 mm
A detailed product specification is available on request.							

WARNINGS and IMPORTANT INFORMATION

Do not use if sterility is compromised or if the packaging is damaged

CAUTION: Do not use damaged products!

Do not reuse

Do not resterilize

CAUTION: Not suitable for reuse, as the required features may fail and cause an infection of the patient.

CAUTION: The instructions for use apply only to the enclosed products.

CAUTION: Open peel pouch at top seam!

Store in a dry place

Keep away from direct sunlight

CAUTION: Using a device cover may impair the quality of the image.

CAUTION: Ventilation openings should not be covered under any circumstances! Covered devices may overheat.

CAUTION: To be used only by healthcare professionals!

CAUTION: Any serious incident that has occurred in relation to the device has to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

OTHER SYMBOLS USED

Sterilized using irradiation

Use-by date

Latex-free

Consult instructions for use. The latest instructions for use can be downloaded at www.heisig.de

Number of consumption units per peel pouches

Personnel number of the manufacturing employee

Batch code
JJMM0123

Manufacturer

Number of peel pouches in a carton

Date of manufacture

Article number

Pharmacy central number

The product is compliant with valid legal requirements for medical devices. Manufacturer is monitored by the notified body with number 0197.

Medical device

Single *sterile* barrier system

Single *sterile* barrier system with protective packaging outside

Environmentally compatible if properly disposed of. No toxic gases arise from burning. Dump without groundwater pollution.

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