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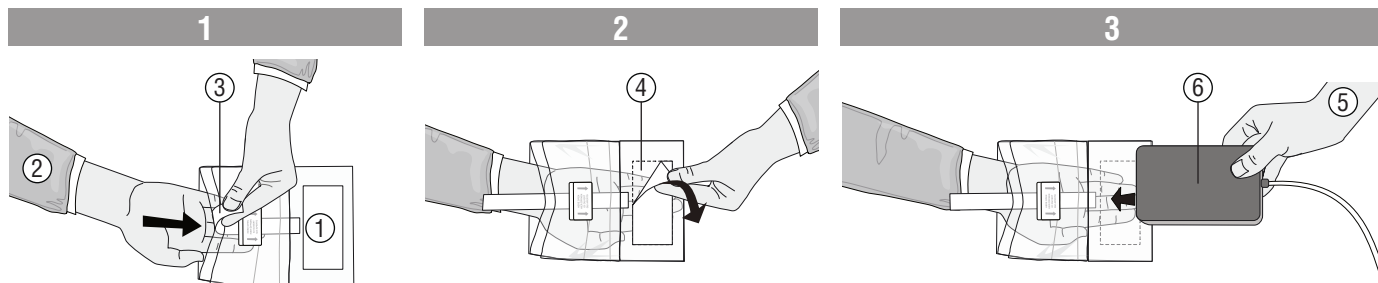
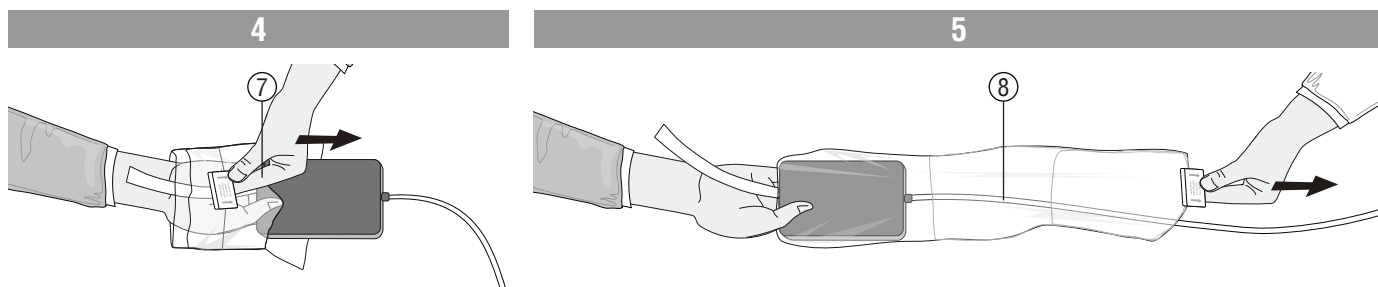
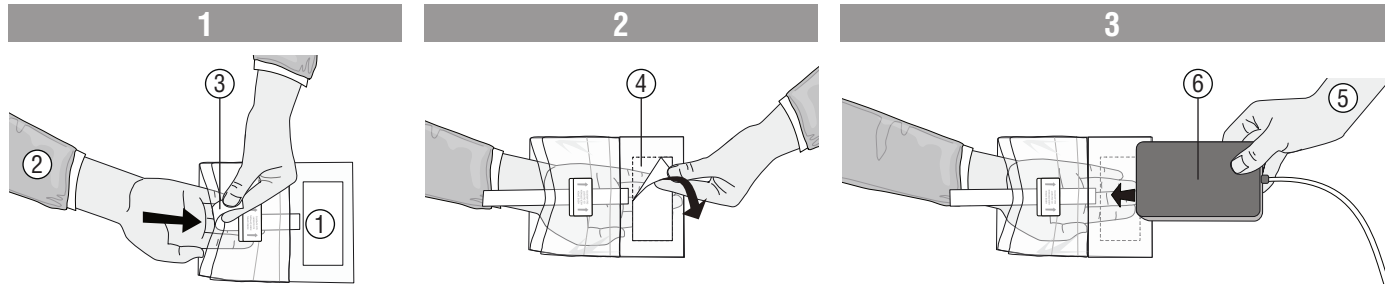
STERINORM® Bag with bottom seam

STERILE R


Intended purpose: Sterile cover for covering equipment for the duration of one application

Place the cover with the adhesive surface (1) upwards – the sterile person (2) reaches underneath the adhesive strip (3) into the interior of the cover.


Using the free hand, the protective paper (4) is pulled forwards off the adhesive surface. The flap of the protective paper that is detached from the adhesive surface is used for this purpose.

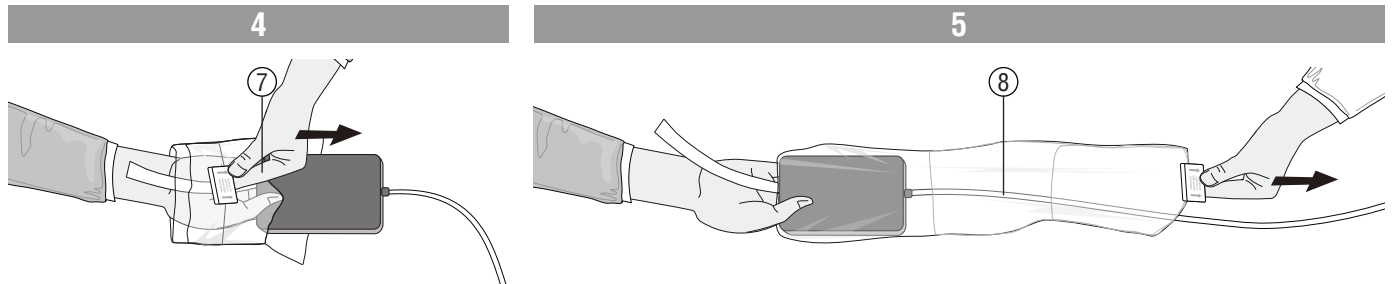The assisting person (5) places the device (6) on the adhesive surface – the sterile person smooths down the adhesive surface from the inside of the cover. If fitted correctly, the product adheres to the device without leaving creases. If any creases are formed, it is possible to reposition the cover on the device.


The sterile person holds the device inside the cover, grips the flap (7) with the free hand and pulls the cover over the device.

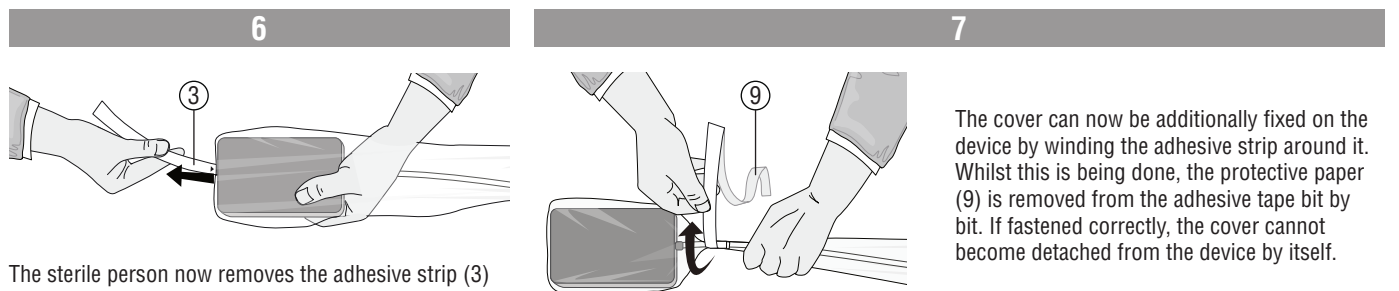Now the assisting person can take over and pull the cover along the cable (8) to the end.

The sterile person now removes the adhesive strip (3) on the exterior of the cover.

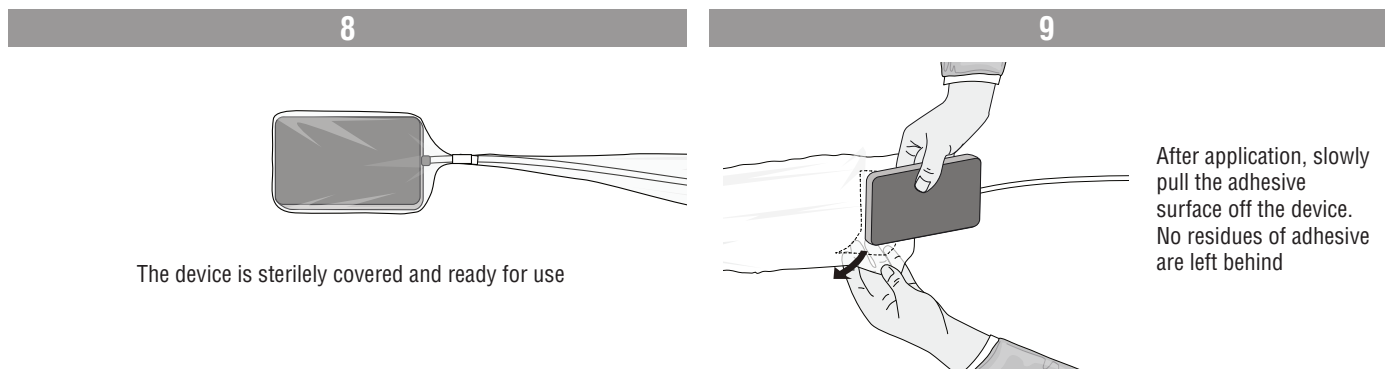The cover can now be additionally fixed on the device by winding the adhesive strip around it. Whilst this is being done, the protective paper (9) is removed from the adhesive tape bit by bit. If fastened correctly, the cover cannot become detached from the device by itself.

The device is sterilely covered and ready for use

After application, slowly pull the adhesive surface off the device. No residues of adhesive are left behind

PACKAGE CONTENT		DETAILS
REF 30100	STERINORM® Bag with bottom seam	Size: 18 x 100 cm
A detailed product specification is available on request.		

WARNINGS and IMPORTANT INFORMATION

Do not use if sterility is compromised or if the packaging is damaged

CAUTION: Do not use damaged products!

Do not reuse

Do not resterilize

CAUTION: Not suitable for reuse, as the required features may fail and cause an infection of the patient.

CAUTION: The instructions for use apply only to the enclosed products.

CAUTION: Open peel pouch **at top seam!**

Store in a dry place

Keep away from direct sunlight

CAUTION: Using a device cover **may impair the quality of the image.**

CAUTION: Ventilation openings should not be covered under any circumstances! Covered devices may overheat.

CAUTION: To be used **only by healthcare professionals!**

CAUTION: Any **serious incident** that has occurred in relation to the device has to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

OTHER SYMBOLS USED

Sterilized using irradiation

Use-by date

Latex-free

www.heisig.de

Consult instructions for use. The latest instructions for use can be downloaded at www.heisig.de

Number of consumption units per peel pouches

Personnel number of the manufacturing employee

Batch code
JMMM0123

Manufacturer

Number of peel pouches in a carton

Date of manufacture

Article number

PZN: 0000000

Pharmacy central number

The product is compliant with valid legal requirements for medical devices. Manufacturer is monitored by the notified body with number 0197.

Medical device

Single *sterile* barrier system

Single *sterile* barrier system with protective packaging outside

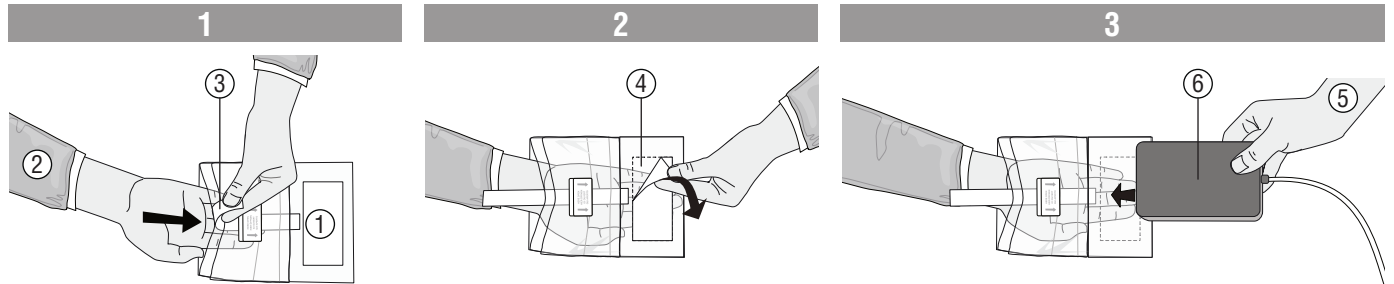
Environmentally compatible if properly disposed of. No toxic gases arise from burning. Dump without groundwater pollution.

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